

# CIRCULAR STAPLER ANVIL SIZER SET

Catalog No. ANS-10

## DESCRIPTION:

The CIRCULAR STAPLER ANVIL SIZER SET includes (1) 25 mm, (1) 29 mm and (1) 33 mm anvil sizers.

## INDICATIONS:

The CIRCULAR STAPLER ANVIL SIZER SET is intended to aid in selecting the appropriate size stapler for use in a variety of applications where a circular intraluminal stapler is used.

## CONTRAINDICATIONS:

None known

## INSTRUCTIONS FOR USE:

Size the proximal bowel segment by inserting the anvil sizers into the end of the open bowel. Insert the anvil sizers obliquely, and then straighten to align the axis of the sizers with the axis of the bowel to determine fit.

## WARNING:

**Excessive force could result in damage to tissue.**

## CLEANING:

Remove gross contamination by hand with mechanical aid (e.g. brush) in submerged water with continuous flow. Wash in automatic washer-disinfection unit. Use washer settings and detergent as instructed by washer-disinfection unit manufacturer. Wear protective gloves and goggles during cleaning.

## STERILIZATION:

Steam Sterilize per ANSI/AAMI ST79:2006 "Comprehensive Guide to Steam Sterilization and Sterility Assurance in Health Care Facilities" using ONE of the following methods:

Method	Steam (Wrapped)	Steam (Wrapped)	FlashSteam (Unwrapped)
Cycle	Gravity	Prevacuum	Gravity
Temperature	270°F/132° C	270°/F132° C)	270° F/132° C
Exposure	15 mins	5 mins	3 mins



**We make it simple, by design.**

Tools for Surgery, LLC  
1339 Stony Brook Road ▪ Stony Brook, New York 11790  
PH (631) 444-4448 ▪ FX (631) 444-5330

[www.toolsforsurgery.com](http://www.toolsforsurgery.com)