

EASY LOOP® Ostomy Rod

Catalog No. EZL-01

INDICATIONS:

The Easy Loop® is used to support an exteriorized loop of bowel when forming a loop colostomy or ileostomy. Use with any size bowel and any size appliance.

INSTRUCTIONS FOR USE:

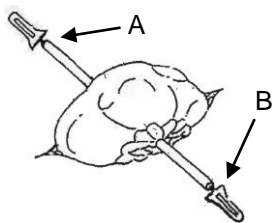


Fig. 1

Inserting the Easy Loop®

Pass the Easy Loop® through the mesentery of the exteriorized bowel (Fig.1).

Apply the stoma appliance wafer over the wound and under the rod.

Fold up and interlock the coupling members by inserting one through the slot of the other (Fig.2).

The device assumes a horseshoe shape when closed. Maneuver the device so that a straighter section of the rod is under the bowel and the device lies flatter, in order to diminish bulging of the pouch.

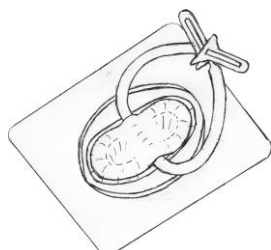


Fig. 2

Removing the Easy Loop®

Cut at point "A" or "B" and remove the device.

Rx only

Latex-free

This product contains DEHP

U.S. Patent No. 6,716,209

EASY LOOP is a registered trademark of Tools for Surgery, LLC

STERILE

EO



Single Use

TfS Tools
for
Surgery

We make it simple, by design.

Tools for Surgery, LLC
1339 Stony Brook Road ▪ Stony Brook, New York 11790
PH (631) 444-4448 ▪ FX (631) 444-5330

www.toolsforsurgery.com