

CIRCULAR STAPLER HEAD SIZER AND POOP SCOOP SET

Catalog No. HSS-10

DESCRIPTION:

The CIRCULAR STAPLER HEAD SIZER AND POOP SCOOP SET includes (1) 25 mm, (1) 29 mm and (1) 33 mm head sizers.

INDICATIONS:

The CIRCULAR STAPLER HEAD SIZER AND POOP SCOOP SET is intended to aid in selecting the appropriate size stapler for use in a variety of applications where a curved circular intraluminal stapler is used. The sizers are also useful in removing feces from the unprepped or poorly prepped bowel prior to inserting a stapler.

CONTRAINDICATIONS:

None known

INSTRUCTIONS FOR USE:

1. Dilate the anus.
2. Size the distal bowel segment by passing head sizers through the anus into the appropriate position in the rectal or rectosigmoid stump.
3. Use the sizer to capture and remove feces prior to inserting a stapler.

WARNING:

Excessive force could result in damage to tissue.

CLEANING:

Remove gross contamination by hand with mechanical aid (e.g. brush) in submerged water with continuous flow. Wash in automatic washer-disinfection unit. Use washer settings and detergent as instructed by washer-disinfection unit manufacturer. Wear protective gloves and goggles during cleaning.

STERILIZATION:

Steam Sterilize per ANSI/AAMI ST79:2006 "Comprehensive Guide to Steam Sterilization and Sterility Assurance in Health Care Facilities" using ONE of the following methods:

Method	Steam (Wrapped)	Steam (Wrapped)	FlashSteam (Unwrapped)
Cycle	Gravity	Prevacuum	Gravity
Temperature	270°F/132° C	270°/F132° C)	270° F/132° C
Exposure	15 mins	5 mins	3 mins



We make it simple, by design.

Tools for Surgery, LLC
1339 Stony Brook Road • Stony Brook, New York 11790
PH (631) 444-4448 • FX (631) 444-5330

www.toolsforsurgery.com