


BARIATRIC ANASTOMOSIS LEAK TESTER

Catalog No. ALT-BA

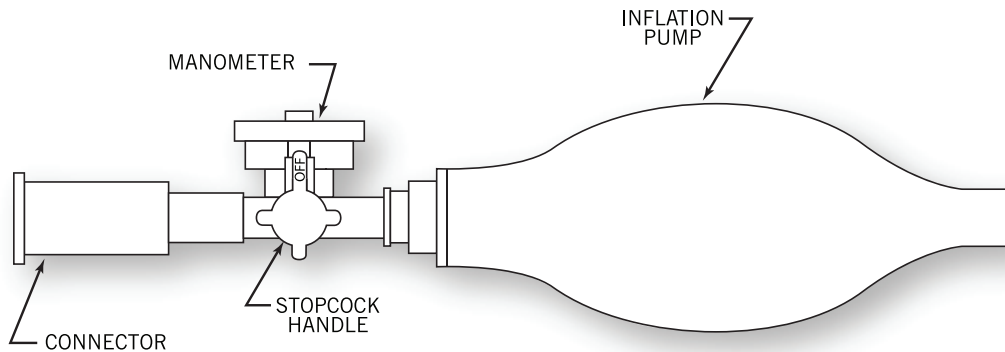

INDICATIONS:

For intraoperative leak testing of a gastrojejunal anastomosis.

INSTRUCTIONS FOR USE:

TO TEST AN ANASTOMOSIS:

1. Immerse the gastrojejunal anastomosis in a pool of saline.
2. Attach the BARIATRIC ANASTOMOSIS LEAK TESTER to a gastric tube via the CONNECTOR.
3. Rotate the STOPCOCK HANDLE so that the OFF label points to the MANOMETER.
4. Compress the INFLATION PUMP slowly and repetitively to inflate the bowel and distend the gastrojejunal anastomosis.
5. Periodically rotate the STOPCOCK HANDLE so that the OFF label points toward the INFLATION PUMP, and read the intraluminal gastrojejunal pressure from the MANOMETER.
6. Rotate the STOPCOCK HANDLE back, so that the OFF label again points to the MANOMETER, to continue inflation.
7. Continue to compress the INFLATION PUMP until the desired intraluminal pressure is reached.
8. Observe the pool of saline for bubbles, which indicate an air leak.
9. Repair the anastomosis if necessary and repeat the leak test.



SINGLE
USE ONLY

Rx only

Clean - not sterile

Latex-free

TfS Tools
for
Surgery

We make it simple, by design.

Tools for Surgery, LLC
1339 Stony Brook Road ■ Stony Brook, New York 11790
PH (631) 444-4448 ■ FX (631) 444-5330

www.toolsforsurgery.com