


# TfS™ LINEAR STAPLER SIZER SET

Catalog No. LSS-01

## DESCRIPTION:

TfS™ LINEAR STAPLER SIZER SET includes 7 stainless steel sizers.

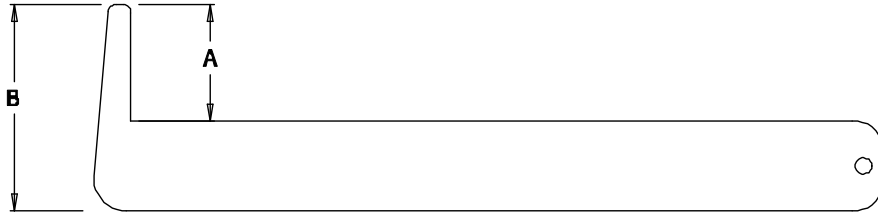

Sizer	Sizes for:	A (mm)	B (mm)
T-30	TL30, TX30, TA30	32	57
T-45	TA45	44	66
T-60	TL60, TX60, TA60	60	85
T-90	TL90, TA90	92	125
CCS	CONTOUR Curved Cutter Stapler	40	62
A-30	PI30, ROTICULATOR 30	34	60
A-55	PI55, ROTICULATOR 55, ACCESS 55	55	81

## INDICATIONS:

TfS™ LINEAR STAPLER SIZERS are intended to aid in selecting the appropriate size linear stapler when the stapler desired for use has a staple line perpendicular to the shaft of the instrument (e.g., Ethicon Endo-Surgery's TL30). These sizers are of particular use when access to the structure to be stapled is limited, for example, when stapling across the rectum in the pelvis during a low anterior resection.

## CONTRAINDICATIONS:

None known.

## INSTRUCTIONS FOR USE:

1. Choose a sizer with a preferred A dimension (staple line length).
2. Try to position the sizer adjacent the structure to be stapled.
3. If the sizer can be positioned, the corresponding staplers may be used.

Patent pending

TfS is a trademark of Tools for Surgery, LLC

Tools for Surgery, LLC  
1339 Stony Brook Road  
Stony Brook, New York 11790  
PH (631) 444-4448  
FX (631) 444-5330  
[www.toolsforsurgery.com](http://www.toolsforsurgery.com)