

IRRIGUT® RECTOSIGMOID IRRIGATOR - RSI™


STERILE	EO
---------	----

For intraoperative rectosigmoid lavage



Single Use

Catalog No. RSI-01

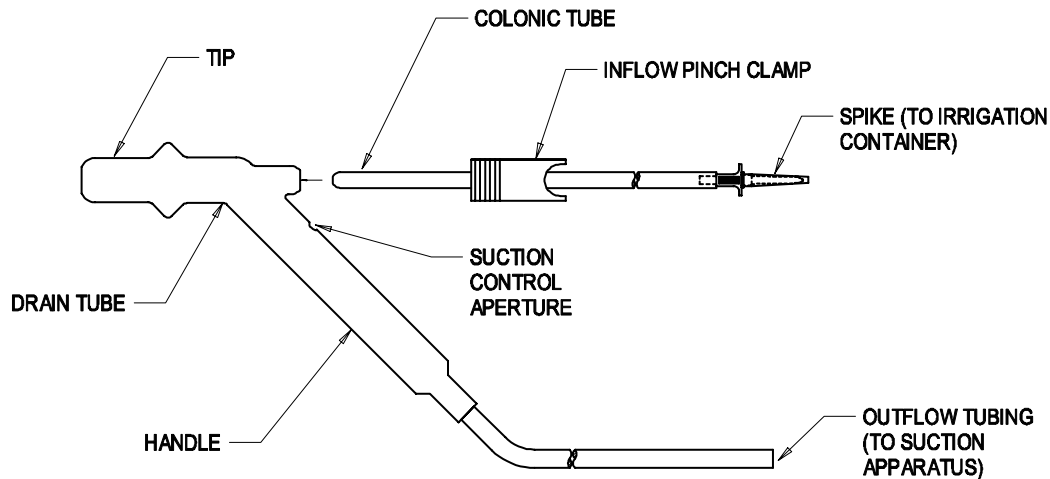

INSTRUCTIONS FOR USE:

1. Warm irrigation fluid to body temperature. Antiseptic or cytotoxic agents may be added to the irrigation fluid.
2. Close the INFLOW PINCH CLAMP.
3. Remove the spike cap and insert the SPIKE into a plastic irrigation container. Use a spike adapter as necessary.
4. Prime the inflow tubing by opening the INFLOW PINCH CLAMP until the COLONIC TUBE is filled.
5. Attach the end of the OUTFLOW TUBING to suction apparatus and turn on suction.
6. Insert the COLONIC TUBE into the DRAIN TUBE.
7. Lubricate DRAIN TUBE TIP and insert into the anus of patient until the expanded portion abuts the anal verge.
8. Carefully advance the COLONIC TUBE into the bowel.
9. Open the INFLOW PINCH CLAMP.
10. Observe the bowel. Close the INFLOW PINCH CLAMP when the desired distension is achieved.
Do not overdistend the bowel.
11. Occlude the SUCTION CONTROL APERTURE on the DRAIN TUBE HANDLE to apply suction to the rectum.
12. Observe the quality of the effluent in the OUTFLOW TUBING.
13. Repeat inflow and outflow cycle (steps 9-11) as required.
14. Remove the device from the anus, drain all residual fluid from the device into suction apparatus, and discard.

Latex and DEHP free
Rx only

Patent Pending
IRRIGUT is a registered trademark of Tools for Surgery, LLC

Manufactured by:
Tools for Surgery, LLC
1339 Stony Brook Road
Stony Brook, New York 11790
PH (631) 444-4448
FX (631) 444-5330
www.toolsforsurgery.com

MADE IN USA