


# BARIATRIC AIR SYRINGE

Catalog No. ALT-BA

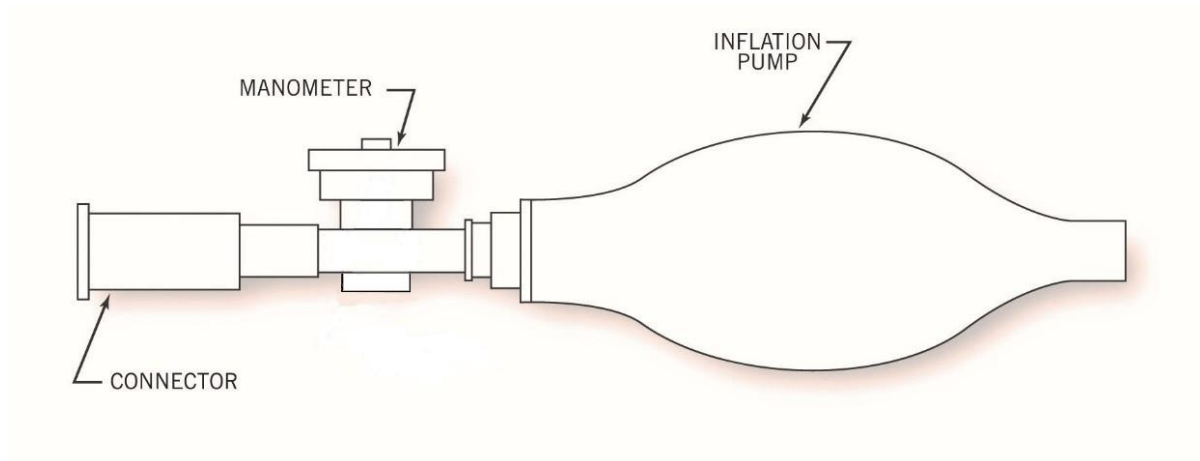

## INDICATIONS:

For intraoperative leak testing of a gastrojejunal anastomosis or gastric closure.

## INSTRUCTIONS FOR USE:

TO TEST A GASTROJEJUNAL ANASTOMOSIS OR GASTRIC CLOSURE:

1. Immerse the anastomosis or closure in a pool of saline.
2. Attach the BARIATRIC AIR SYRINGE to a gastric tube via the CONNECTOR. Alternatively, remove the CONNECTOR and attach the male Luer slip directly to the gastric tube.
3. Compress the INFLATION PUMP slowly and repetitively to inflate the bowel.
4. Observe the MANOMETER and inflate until the desired intraluminal pressure is reached.

**DO NOT OVERINFLATE. OVERINFLATION MAY DAMAGE OR RUPTURE A SUTURE OR STAPLE LINE.**

5. Observe the saline pool for bubbles, which indicate an air leak.
6. Repair the anastomosis or gastric closure if necessary and repeat the leak test.



Rx only

Clean - not sterile

**TfS** Tools  
for  
Surgery

**We make it simple, by design.**

Tools for Surgery, LLC  
8 Technology Drive, #100 ▪ East Setauket, New York 11733  
PH (631) 444-4448 ▪ FX (631) 444-5330

[www.toolsforsurgery.com](http://www.toolsforsurgery.com)