


BARIATRIC AIR SYRINGE

Catalog No. ALT-BA

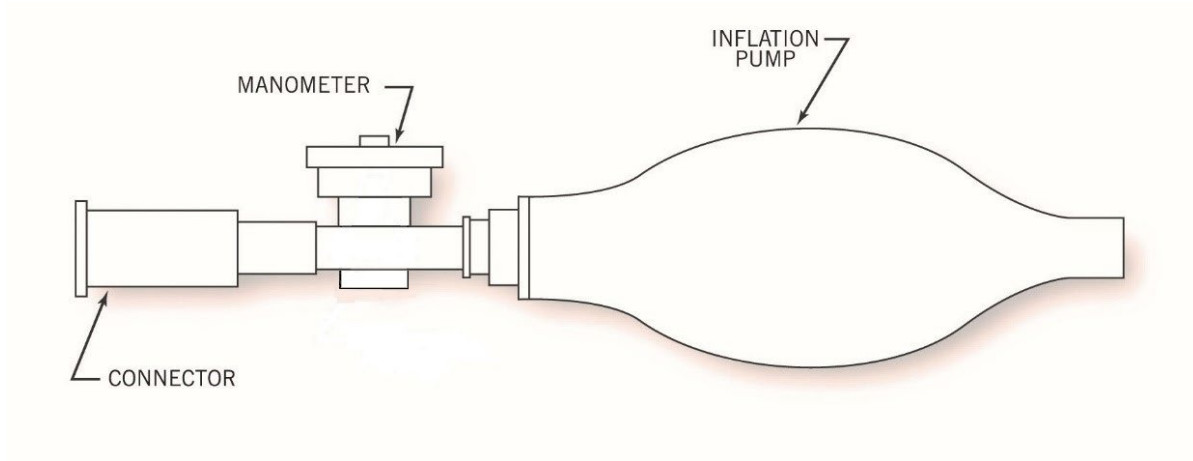

INDICATIONS:

For gastrointestinal insufflation.

INSTRUCTIONS FOR USE:

1. Attach the BARIATRIC AIR SYRINGE to an orogastric tube via the CONNECTOR.
Alternatively, remove the CONNECTOR and attach the male Luer slip directly to the orogastric tube.
2. Compress the INFLATION PUMP slowly and repetitively to inflate the bowel.
3. Observe the MANOMETER and inflate until the desired intraluminal pressure is reached.

DO NOT OVERINFLATE. OVERINFLATION MAY CAUSE GASTROINTESTINAL INJURY.



Rx only

Clean - not sterile

TfS Tools
for
Surgery

We make it simple, by design.

Tools for Surgery, LLC
8 Technology Drive, #100 ▪ East Setauket, New York 11733
PH (631) 444-4448 ▪ FX (631) 444-5330

www.toolsforsurgery.com